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HS-IFU-702 VER 02 (2019.02.05)

## 1. Geral

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HS-IFU-702 VER 02 (2019.02.05)

## 1. Overview

- Product Category : Feldspatic ceramic leucite Block;
- Product Name : Rosetta BM;
- Intended Use of the Device: This product is used to make artificial tooth in order for damaged tooth to get its function and beauty restored;
- Packaging Unit : Refer to HASS standard package.

## 2. Instructions for use

### (1) Preparation for use

Check the defects of the product for any damage or crack before use.

## 3. Cautions

### (1) CAUTIONS BEFORE USE

- Check the defects of the product for any damage or crack before use.
- Be careful not to damage the milling tool of the CAD/CAM machine when attaching or detaching the product.
- Be careful not to get your hand caught in the milling tool.
- The jig should be attaching to an accurate location.
- Suppress or remove the dust which may occur during the operation of CAD/CAM machine.
- Do not drop the product on the ground or apply heavy force as it may damage the product.
- Keep the product out of reach of infants and children.

### (2) STORAGE AND MAINTENANCE BEFORE USE

- Store the product at room temperature in a dry place.
- Pack and store the product properly to ensure that it is not damaged.
- Store the product at temperatures ranging from 0°C~40°C, in combination with relative humidity of 10% r.H ~ 90% r.H, under atmospheric pressures ranging from 500 hPa ~ 1060 hPa.

## 4. Mechanical and Physical Properties

- Material : Feldspatic ceramics leucite.
- Flexural Strength : over 120 MPa.
- Chemical Solubility : below 100 ug/cm<sup>2</sup>.
- Coefficient of Thermal Expansion : 10.0 (±0.5) x 10<sup>-6</sup>K<sup>-1</sup>

\*This is a single use product \*Do not reuse.

## 5. Pictograph

	Do not reuse		Authorized Representative in the European Community
	Batch Code		Catalogue Number
	Manufacturer		Do not use if package is damaged
	Caution		Non Sterile
	Date of Manufacture		Consult Instructions for Use

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