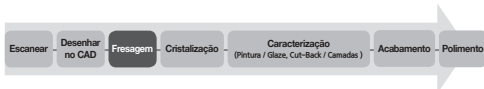
1. GERAL

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1. Overview

- Product Category : Glass-ceramic Block
- Product Name : Rosetta SM
- Intended Use of the Device : This product is used to make artificial tooth in order for damaged tooth to get its function and beauty restored.
- Packaging Unit : Refer to HASS standard package.

2. Instruction

3. Cautions

(1) Cautions before Use

- ① Check the defects of the product for any damage or crack before use.
- ② Be careful not to damage the milling tool of the CAD/CAM machine when attaching or detaching the product.
- ③ Be careful not to get your hand caught in the milling tool.
- ④ The jig should be attaching to an accurate location.
- ⑤ Suppress or remove the dust which may occur during the operation of CAD/CAM machine.
- ⑥ Do not drop the product on the ground or apply heavy force as it may damage the product.
- ⑦ Keep the product out of reach of infants and children.

(2) Storage and Maintenance before Use

- ① Store the product at room temperature in a dry place.
- ② Pack and store the product properly to ensure that it is not damaged.
- ③ Store the product at temperatures ranging from 0°C ~ 40°C, in combination with relative humidity of 10% r.H ~ 90% r.H, under atmospheric pressures ranging from 500 hPa ~ 1060 hPa.

4. Mechanical and Physical Properties

- ① Material : Glass-ceramics
- ② Flexural Strength : over 300 MPa
- ③ Chemical Solubility : below 100 µg/cm²
- ④ Coefficient of Thermal Expansion : 10.0 (±0.5) x 10⁻⁶ K⁻¹

* This is a single-use product. * Do not reuse.

5. Pictograph

Do not reuse	Caution	Catalogue Number	Consult instructions for Use
Batch Code	Date of Manufacture	Do not use if package is damaged	
Manufacturer	Authorized Representative in the European Community	Non Sterile	

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① Caution

There may be a slight difference between the displayed temperature and the actual temperature of each furnace. Before you sinter Rosetta® SM blocks, please verify that the above recommended schedule is suitable for the furnace being used. Otherwise, try to find the optimized crystallization temperature through the following process.

- If there are some changes in the shape of restoration during crystallization heat treatment, please reduce the final temperature by 10°C ~ 15°C.